

# PRIME|ONE EXTRA GRIP LATEX GLOVES

SINGLE USE DISPOSABLE LATEX EXAMINATION GLOVES

## CONFIGURATION OVERVIEW

|                  |  |
|------------------|--|
| <b>Features</b>  | Cuff strengthened by a rolled cuff<br>Easy donning and removal<br>Excellent tactile sensibility (Micro textured)<br>Suitable for short term food contact |
| <b>Shape:</b>    | Ambidextrous   |
| <b>Material:</b> | Latex  |
| <b>Type:</b>     | Non-Sterile<br>Powder-free<br>Thiuram Free<br>Polymer Coated   |



## TECHNICAL SPECIFICATIONS

|                             |                         |       |
|-----------------------------|-------------------------|-------|
| <b>Physical Properties:</b> | Elongation at Break (%) | ≥500% |
|                             | Force Break (N)         | ≥6N   |

|                                   |                  |
|-----------------------------------|------------------|
| Location of Thickness Measurement | Single Wall (mm) |
| Finger Tip                        | ≥ 0.13 ±0.03     |
| Palm                              | ≥ 0.12 ±0.03     |

|                          |                           |
|--------------------------|---------------------------|
| <b>Available Length:</b> | ≥ 240 mm                  |
| <b>Width (± 5 mm) :</b>  | XS: 75 S: 85 M: 95 L: 106 |
| <b>Weight M size:</b>    | 5.8 g ± 0.2 g             |
| <b>Shelf life:</b>       | 5 years                   |

## QUALITY ASSURANCE

**Class I Medical Device according to EU REG. MDR 2017/745**  
**Class III Personal Protective Equipment according to EU REG. 2016/425**

**Conform to** EN 455-1:2020+A2:2024, EN 455-2:2024, EN 455-3:2015, EN 455-4:2009, EN ISO 14971:2019/Amd.11:2021, EN ISO 15223-1:2021/Amd.1:2025, EN ISO 20417:2021, EN ISO 374-1:2016+A1:2018, EN ISO 374-2:2019, EN ISO 374-4:2019 and EN ISO 374-5:2016, EN ISO 21420:2020, EN ISO 10993-1:2025, EN ISO 10993-5:2009, EN ISO 10993-10:2010

Conform to : EC REG. 1935/2004

## PRODUCT VARIANTS

|         |                                |
|---------|--------------------------------|
| Art-No. | GLOCL60-AKZ-WH-XS1/S01/M01/LL1 |
| Color   | White                          |
|         | ○                              |

**Available Sizes:** XS S M L



**PACKAGING**

|                   |   |
|-------------------|---|
| Pcs/Dispenser:    | 100   |
| Dispenser/Carton: | 10  |
| Carton Size:      | 36,5 x 25 x 23 cm                           |
| Carton weight:    | XS: 6 Kg. S: 6,5 Kg. M: 7,5 Kg. L: 8,00 Kg. |
| Pallet size:      | 80 x 120 x 176 cm                           |
| Pallet weight :   | XS: 398 Kg. S: 430 Kg. M: 492 Kg. L: 524Kg. |
| Carton / layer:   | 9   |
| Layer / pallet:   | 7   |
| Carton / pallet:  | 63  |

**PRODUCT INSTRUCTIONS FOR USE**

- 1) Before usage, inspect the gloves for any defects or imperfections.
- 2) **Usage is for single use only.** If gloves are reused, the risk of contamination and infection increases due to improper cleaning processes, as well as an increased risk of holes and tears during reuse caused by weakening of the gloves.
- 3) Sizing: Select the right size of glove for your hands.
- 4) Donning: Hold the glove by the cuff with one hand. Align the glove thumb with your other hand's thumb and slide your hand into the glove, inserting one finger into each glove finger. Pull on the glove palm to achieve a good fit. Don the other glove using the same procedure.
- 5) Inspection: punctures or tears may occur after donning. Inspect the glove after donning and stop using them immediately if any damage is found.
- 6) Doffing: Hold the glove cuff and pull towards the fingers until the glove comes off.
- 8) Disposal: For the proper disposal of all used gloves, please follow your institution's policies for disposal

**STORAGE:** Store in cool, dry place at room temperature. Opened box should be shielded from exposure to direct sun or fluorescent/ultraviolet light sources. Storage above 30° will lead to accelerated aging and should be avoided.

**WARNING:**

Potentially allergenic ingredients: **NATURAL RUBBER LATEX.**



Rev. 0\_27-02-2026

